## **Instructions for Use**

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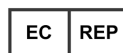
CE-mærkning

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Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Exact Diagnostics LLC  
3400 Camp Bowie Blvd CBH-214  
Fort Worth Texas 76107  
United States

EDX SARS-CoV-2 Positive Run Control Summary	
Catalogue number	COV019CE
Analytes	SARS-CoV-2 RNA Transcripts: Envelope (E) Gene, Nucleocapsid (N) Gene, ORF1ab Gene, RdRP Gene, Spike Protein (S) Gene Human Genomic DNA
Matrix	Synthetic Matrix
Preservative	Sodium azide
Storage	-20°C or below
Fill volume	0.3mL
Number of vials per kit	5 vials
Number of uses	Single use
Shelf life	12 months from date of manufacture
Precautions	Biological risks
Regulatory status	For in vitro diagnostic use

**INTENDED USE:**

EDX SARS-CoV-2 Positive Run Control is an external quality control intended to be used with SARS-CoV-2 assays to monitor the performance of that given assay. Routine use of EDX SARS-CoV-2 Positive Run Control allows laboratories to evaluate day-to-day and lot-to-lot variation of their molecular assay and test for operator proficiency.

**PRODUCT DESCRIPTION:**

EDX SARS-CoV-2 Positive Run Control is manufactured with synthetic RNA transcripts containing five gene targets (E, N, ORF1ab, RdRP, and S Genes of SARS-CoV-2) and Human Genomic DNA. The control is intended to test the entire process of a molecular assay including extraction, amplification, and detection.

EDX SARS-CoV-2 Positive Run Control is internally value assigned using Bio-Rad Digital Droplet PCR (ddPCR). EDX SARS-CoV-2 Positive Run Control is formulated in Synthetic Matrix, which contains preservatives.

**PROCEDURE:**

EDX SARS-CoV-2 Positive Run Control requires an extraction step. The control must be treated in a similar manner to other tested samples. This allows the operator to assess their extraction technique.

EDX SARS-CoV-2 Positive Run Control is an unassayed control and does not have assigned values. The control is not a substitute for the manufacturer's kit controls provided with the assay. EDX SARS-CoV-2 Positive Run Control is to be tested according to the assay manufacturer's or testing laboratory's instructions and recommendations. Expected results using EDX SARS-CoV-2 Positive Run Control must be established by end user for their specific assay.

**STORAGE AND HANDLING:**

EDX SARS-CoV-2 Positive Run Control should be stored at -20°C or below.

Thaw EDX SARS-CoV-2 Positive Run Control at room temperature and vortex briefly prior to use.

Once thawed, EDX SARS-CoV-2 Positive Run Control is stable for 24 hours when stored at 2-8°C. Any remaining materials must be disposed of after one use. Do not reuse.

Do not use EDX SARS-CoV-2 Positive Run Control beyond the expiration date.

**LIMITATIONS:**

EDX SARS-CoV-2 Positive Run Control should not be diluted without appropriate validation. Laboratories can validate use of the EDX SARS-CoV-2 Negative Run Control as a suitable diluent for this product.

For in vitro diagnostic use.

**WARNINGS AND PRECAUTIONS:**

EDX SARS-CoV-2 Positive Run Control contains synthetic RNA transcripts but should be considered biohazardous. Universal precautions and proper disposal should be practised<sup>1</sup>. Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

Discard product if packaging is damaged or leaking. Disinfect liquids, materials or spills with a 0.5% sodium hypochlorite solution. Dispose of all materials and liquids used in the procedure as if they contained pathogenic agents.

**REFERENCES:**

<sup>1</sup> Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee. 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Health Care Settings. *Am J Infect Control* 2007 Dec, 35 (10 Suppl 2): S65-164.

BIO-RAD and DROPLET DIGITAL PCR are trademarks of Bio-Rad Laboratories, Inc. in certain jurisdictions.

**SYMBOL REFERENCES:**

Catalogue number

Lot number

For in vitro diagnostic use

Biological risks

Expiration date

Upper limit of temperature

Manufacturer

Caution

Consult instructions for use

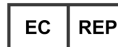
Positive control

European Mark of Conformity

Authorised representative

Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Exact Diagnostics LLC  
3400 Camp Bowie Blvd CBH-214  
Fort Worth Texas 76107  
United States

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BIO-RAD och DROPLET DIGITAL PCR är varumärken från Bio-Rad Laboratories, Inc. i vissa rättskipningsområden.