



SaSelect™ medium

Direct Identification Visibly Reliable



SaSelect™ Optimum Sensitivity within 18-24 hours

- 322 various clinical specimens: suppurations, wounds, respiratory secretion, nasal swab, blood culture...
- 152 confirmed *Staphylococcus aureus* isolated on at least one medium: SaSelect™ and Mannitol Salt Agar.


	SaSelect™ 24 hr	MSA 24 hr + coagulase test	MSA 48 hr + coagulase test
Sensitivity	98.0 % (149/152)	90.8 % (138/152)	94.7 % (144/152)
Specificity	99.9 %	99.4 %	99.4 %

Staphylococcus aureus on: MSA ⇒ Mannitol positive colony confirmed by coagulase test
 SaSelect ⇒ Pink to orange colonies

→ The Direct Identification: *Staphylococcus aureus*



→ Differentiation of Other Staphylococci

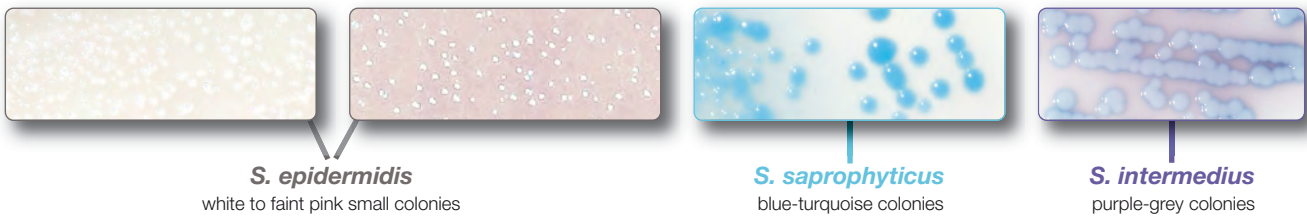

→ Visualization of Mixed Cultures



Ordering Information

Catalog No.	Description
63748	SaSelect™20 plates 90mm
63747	MRSASelect™20 plates 90mm



Bio-Rad
Laboratories

For further information, please contact the Bio-Rad office nearest you or visit our website at www.bio-rad.com/diagnostics

Clinical
Diagnostics Group

Website www.bio-rad.com/diagnostics U.S. 1-800-2BIO-RAD Australia 61-2-9914-2800 Austria 43-1-877-8901 Belgium 32-9-385-5511 Brazil 5521-3237-9400
 Canada 1-514-334-4372 China 86-21-64260808 Czech Republic 420-241-430-532 Denmark +45-4452-1000 Finland 358-9-804-22-00 France 33-1-47-95-60-00
 Germany +49-(0)89-318-840 Greece 90-210-7774396 Hong Kong 852-2789-3300 Hungary +36-1-459-6100 India 1-800-180-1224 Israel 972-3-9636050 Italy +39-02-216091
 Japan 81-3-6361-7070 Korea 82-2-3473-4460 Mexico +52 (55) 5488-7670 The Netherlands +31-318-540666 New Zealand 64-9-415-2280 Norway 47-23-38-41-30
 Poland 48-22-3319999 Portugal 351-21-472-7700 Russia 7-495-721-14-04 Singapore 65-6415-3170 South Africa 27-11-442-85-08 Spain 34-91-590-5200
 Sweden 46-8-555-127-00 Switzerland 41-61-717-95-55 Thailand 662-651-8311 United Kingdom +44-(0)20-8328-2000